

▲ Open here

▲ Open here


## Polyp Collection Kit REF BBP-50

【Main Material】 PC, PVC, HDPE

【Indications】

To collect the polyps through the endoscope channel which are greater than 1mm, for pathological analysis.

【Structure】

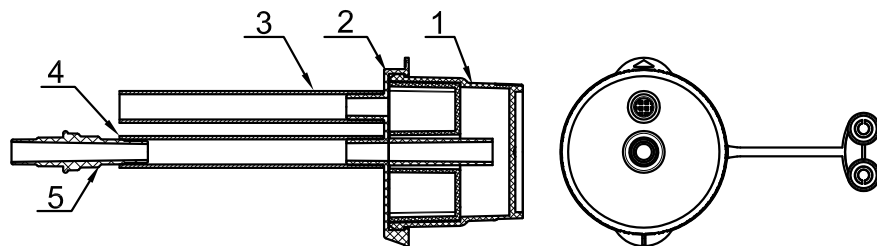

1.Bottle 2.Lid 3.Collection hose 4.Exhaust hose 5.Connector

【User Instruction】

1.Operation

- 1.1.Connect the exhaust hose to the suction unit
- 1.2.Connect the collection hose to the endoscope
- 1.3.Rotate the bottle to make sure the arrow on the lid is aligned with Position 1
- 1.4.To entrap the first polyp into the cell
- 1.5.After the first polyp is trapped,repeat step 1.3 to align arrow with any free position 2,3 and 4 if needed
- 1.6.Repeat step 1.4 and 1.5 for each additional polyp (same patient only)

2.Removal

Open the lid, remove the tissue specimen by using tweezers and other tools from the collection cell.

【Notes】

- 1.Operator must be trained and qualified;
- 2.The product is disposable for one patient;
- 3.Each specimen bottle can take 4 specimen.



1. "LOT NUMBER" SEE SEALING PART
2. SHELF LIFE: 5 YEARS

Rx only



**Manufacturer**

**Micro-Tech (Nanjing) Co., Ltd.**

No.10 Gaoke Third Road, Nanjing National Hi-Tech Industrial

Development Zone, Nanjing 210032, Jiangsu Province, PRC

TEL:+ 86 25 58609879, 58646393

FAX:+ 86 25 58744269

Email: info@micro-tech.com.cn

www.micro-tech.com.cn



**MICRO-TECH**<sup>TM</sup>  
ENDOSCOPY

**Distributed By**

**Micro-Tech USA Inc.**

2855 Boardwalk Drive, Ann Arbor, MI 48104 USA

Tel: 734-259-3768

Toll free: 877-552-4027

info@micro-tech-usa.com



**Shanghai International Holding Corp. Gmbh (Europe)**

Eiffestrasse 80,20537 Hamburg,Germany