


Polyp Collection Kit REF BBP-50-M

【Main Material】 PC, PVC, HDPE

【Indications】

To collect the polyps through the endoscope channel which are greater than 1mm, for pathological analysis.

【Structure】

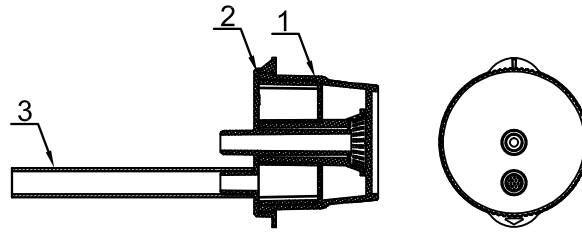

1.Bottle 2.Lid 3.Hose

【User Instruction】

1.Operation

- 1.1.Connect the hose to the endoscope
- 1.2.Rotate the bottle to make sure the arrow on the lid is aligned with Position 1
- 1.3.To entrap the first polyp into the cell
- 1.4.After the first polyp is trapped,repeat step 1.2 to align arrow with any free position 2,3 and 4 if needed
- 1.5.Repeat step 1.3 and 1.4 for each additional polyp (same patient only)

2.Removal

Open the lid, remove the tissue specimen by using tweezers and other tools from the collection cell.

【Notes】

- 1.Operator must be trained and qualified;
- 2.The product is disposable for one patient;
- 3.Each specimen bottle can take 4 specimen;
- 4.Each specimen bottle could be taken out separately if needed.



1. "LOT NUMBER" SEE SEALING PART
2. SHELF LIFE: 5 YEARS

Rx only



Manufacturer

MICRO-TECH (NANJING) CO., LTD.

NO. 10 Gaoke Third Road Nanjing National Hi-Tech Industrial Development Zone, Nanjing 210032, Jiangsu Province, People's Republic of China

TEL:+ 86 25 58609879, 58646393

FAX:+ 86 25 58744269

Email : info@micro-tech.cn

Http:// www.micro-tech.com.cn

Distributed By

Micro-Tech USA

2855 Boardwalk Drive, Ann Arbor, MI 48104 USA

Tel: 734-259-3768

Toll free: 877-552-4027

Email : order@micro-tech-usa.com

http://www.micro-tech-usa.com



MICRO-TECH™
ENDOSCOPY



Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany