


## **Blue Beacon GI Marker 0.4% Syringe Instructions for Use**

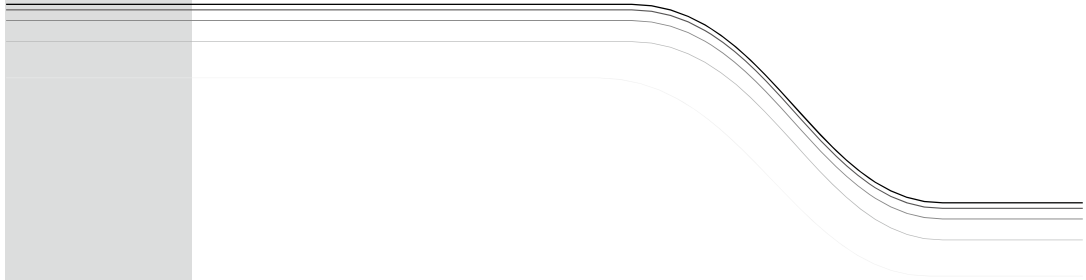





## IMPORTANT INFORMATION

---

Caution: Federal law restricts this device to sale by or on the order of a physician. Read all instructions carefully before use. They contain essential information on using this device safely and effectively. Keep these instructions in a safe, accessible location, as you may need to refer to them again. If you have any questions or comments about any information in these instructions, please contact Micro-Tech.

## Main Components

---

Indigo-5, 5'-Disulfonate, Sodium Pyrosulfite, Water for Injection

## INTENDED USE

---

Blue Beacon™ GI Marker, Indigo Carmine 0.4% is indicated for temporary endoscopic marking of tissue in the GI tract, to aid tissue visualization.

## CONTRAINDICATIONS

---

- Gastrointestinal obstruction or perforation;
- Allergy to device or any of the ingredients in the product.

## POTENTIAL COMPLICATIONS

---

No adverse effects or patient discomfort were reported with Blue Beacon™ GI Marker. However, any inadvertent injection of marker into the intraperitoneal space should be monitored closely for adverse reactions.

## WARNINGS

---

- For use in the GI tract only– NOT FOR INTRAVASCULAR OR INTRAMUSCULAR INJECTION.
- Only the fluid is sterile. Do not use in a sterile or aseptic area without proper precautions.
- **The product is intended for single use only! DO NOT re-use, re-sterilize, and/or reprocess.** Re-use, re-sterilization or reprocessing may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Re-use, re-sterilization or reprocessing may also create a risk of contamination of the device and/ or cause patient infectious disease(s). Contamination of the device may lead to injury, illness or death of the patient. Micro-Tech assumes no liability with respect to instruments reused, re-sterilized or reprocessed .
- Do not use this instrument for any purpose other than its intended use.
- The product is only intended for adult populations.
- This device is not made with natural rubber latex.
- **The instrument is intended for use under the direct supervision of a trained physician only.** A thorough understanding of the technical principles, clinical applications, and associated risks is expected before usage.

**【Product Name】** Blue Beacon™ GI Marker

**【Concentration】** Indigo Carmine: 0.4% +/- 0.075%, dilute before using

**【Packaging】** Packaged in vial or syringe inside a sealed pouch

**【Production Date】** See packaging

**【Sterilization】** Fluid sterilized by steam

**【Shelf Life】** 2 years

## PREPARATION

---

1. Reference the product label and choose the appropriate device.
2. Fluid is supplied STERILE.
3. Open the package carefully after verifying the expiration date.  
CAUTION: For use in the GI tract only–NOT FOR INTRAVASCULAR OR INTRAMUSCULAR INJECTION
4. Sterile unless damaged. Leakage of product maybe evidence of damage. Do not use if damaged.
5. Dilute the solution as required prior to use.

## INSTRUCTIONS FOR USE

---

1. Select a suitable endoscopic injection needle or spray catheter, with a luer lock connection. Prime the catheter prior to use.
2. With the needle retracted, carefully insert the catheter through the biopsy channel of the endoscope under direct visualization until the needle emerges into the endoscopic view.
3. When the needle is properly positioned, advance the needle and inject into the target tissue.

**CAUTION: Do not insert the needle perpendicular to the colon, as this may lead to perforation of the serosa and injection of the marker directly into the peritoneal cavity!**

4. Carefully inject aliquots as directed by the physician.

**CAUTION: Observe for leakage of the marker into the lumen. If this occurs, stop the injection and reposition the needle.**

## STORAGE

---

The product should be stored in the dark, at room temperature.

## PRODUCT DISPOSAL

---

1. Discard any unused product after the syringe/vial has been opened.
2. After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local, and administrative laws and regulations.

## Limited Warranty and Disclaimers

---

1. Limited Warranty to Buyer. Micro-Tech USA warrants to Buyer that, for the earlier of one (1) year from the date of purchase, or until the product is used by Buyer, the products will be free from defects in materials and workmanship when stored and used in accordance with the instructions for storage and use provided by Micro-Tech USA and in accordance with applicable regulatory requirements. Descriptions or specifications appearing in Micro-Tech USA's literature are meant to generally describe the products and do not constitute any express warranties. In the event that Micro-Tech USA gives technical advice with respect to the product, it is agreed that such advice is given without any

liability on Micro-Tech USA's part. Any guarantee of specific properties of or in the products shall only be effective if and to the extent specifically confirmed by Micro-Tech USA in writing. These warranties shall not apply for product failure or deficiency due to improper storage, alteration, or the consequences of uses for which the products were not designed or that adversely affect the products' integrity, reliability, or performance.

2. Disclaimer and Release. **THE WARRANTIES, OBLIGATIONS, AND LIABILITIES OF Micro-Tech USA AS SET FORTH HEREIN ARE EXCLUSIVE. BUYER HEREBY WAIVES ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO THE PRODUCTS AND ANY OTHER GOODS OR SERVICES DELIVERED BY BUYER, INCLUDING, BUT NOT LIMITED TO: (1) ALL OTHER EXPRESS AND IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND (2) ANY IMPLIED WARRANTY ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING, OR USAGE OF TRADE.**
3. Implied Warranties. The purchase of products may be subject to laws in the territories applicable to the sale of the products by Micro-Tech USA to Buyer that impose implied warranties, conditions, or obligations that cannot be excluded, restricted, or modified, or can only be excluded, restricted, or modified to a limited extent. The provisions of Paragraphs 2 and 4 shall apply to the greatest extent allowed by such laws.
4. Limitation of Liability. **EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, Micro-Tech USA'S LIABILITY UNDER THIS WARRANTY IS LIMITED TO: (a) THE REPLACEMENT OF THE PRODUCTS OR THE RE-SUPPLY OF EQUIVALENT PRODUCTS; (b) THE REPAIR OF THE PRODUCTS OR PAYMENT OF THE COST OF REPAIRING THE PRODUCTS; or (c) PAYMENT OF THE COST OF REPLACING THE PRODUCTS OR ACQUIRING EQUIVALENT PRODUCTS. MICRO-TECH USA SHALL HAVE NO OBLIGATION OR LIABILITY, WHETHER ARISING IN CONTRACT (INCLUDING WARRANTY), TORT (INCLUDING ACTIVE, PASSIVE, OR IMPUTED NEGLIGENCE, STRICT LIABILITY, OR PRODUCT LIABILITY) OR OTHERWISE, FOR ANY SPECIAL, CONSEQUENTIAL, PUNITIVE, INCIDENTAL, OR INDIRECT DAMAGES, OR FOR LOSS OF USE, LOSS OF REVENUE, LOSS OF BUSINESS, LOST PROFIT, OR OTHER FINANCIAL LOSS ARISING OUT OF OR IN CONNECTION WITH ANY PRODUCT OR OTHER GOODS OR SERVICES FURNISHED BY MICRO-TECH USA, EVEN IF MICRO-TECH USA WAS AWARE OF THE POSSIBILITY OF SUCH DAMAGES OR LOSS.**





**Manufacturer**


**Micro-Tech (Nanjing) Co., Ltd.**

No.10 Gaoke Third Road, Nanjing National Hi-Tech Industrial  
Development Zone, Nanjing 210032 ,Jiangsu Province, PRC  
TEL:+ 86 25 58609879, 58646393  
FAX:+ 86 25 58744269  
Email: info@micro-tech.com.cn  
www.micro-tech.com.cn;

**Distributed By**

**Micro-Tech USA Inc.**

2855 Boardwalk Drive, Ann Arbor, MI 48104 USA  
Tel: 734-259-3768  
Toll free: 877-552-4027  
info@micro-tech-usa.com

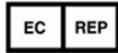

Shanghai International Holding Corp. GmbH(Europe)  
Eiffestrasse 80, 20537 Hamburg Germany

1-0002566 REV: 3  
2018.01.27