

In-Line Suction Polyp Trap

Instructions for Use

In-line Suction Polyp Trap

Instructions for Use

Rx only



IMPORTANT INFORMATION

Caution: Federal law restricts this device to sale by or on the order of a physician. Read all instructions carefully before use. They contain essential information on using this device safely and effectively. Keep these instructions in a safe, accessible location, as you may need to refer to them again. If you have any questions or comments about any information in these instructions, please contact Micro-Tech USA.

INTENDED USE

The single use In-Line Suction Polyp Trap is used for suction retrieval of endoscopically removed polyps.

CONTRAINDICATIONS

Contraindications include, but are not limited to, those specific to any endoscopic procedure.

WARNINGS

1. **The product is intended for single use only! DO NOT re-use, re-sterilize, and/or reprocess.** Re-use, re-sterilization or reprocessing may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Re-use, re-sterilization or reprocessing may also create a risk of contamination of the device and/or cause patient infectious disease(s). Contamination of the device may lead to injury, illness or death of the patient. Micro-Tech USA assumes no liability with respect to instruments reused, re-sterilized or reprocessed.
2. The in-line suction polyp trap has a built-in luer lock that fits standard luer lock syringes (See Figure 2). This allows for the use of irrigation to remove specimens within the specimen screen.
3. When removing a specimen from the trap, make sure that the entire specimen is located within the specimen screen prior to removing trap from system to prevent the loss of retrieved tissue.
4. Do not use this instrument for any purpose other than its intended use.
5. This device is not made with natural rubber latex.
6. **The instrument is intended for use under the direct supervision of a suitably trained physician only.** A thorough understanding of the technical principles, clinical applications, and associated risks is expected before usage.

【 Product Name 】 In-Line Suction Polyp Trap

【 Production Date 】 See packaging


【 Shelf Life 】 3 years

PREPARATION

1. Reference the product label and choose the appropriate device.
2. Inspect the package before use for any damage. Do not use if package is damaged. Open the package carefully after verifying the shelf life.
3. If this device shows any signs of damage, do not use. Do not attempt to repair a nonfunctional or damaged device.

INSTRUCTIONS FOR USE

1. Connect the barbed fitting of the trap to the wall suction (See Figure 1).
2. Connect the tube on the trap to the suction connector (barb) on the endoscope (See Figure 1).
3. Retrieve resected polyp by applying suction via the endoscope suction valve.
4. Remove the trap from the endoscope and the wall suction tubing once the polyp is retrieved.
5. Reattach the wall suction tubing to the suction connector (barb) on the scope.
6. Separate the trap by rotating both halves in opposite (counterclockwise) directions (See Figure 2). Note: After separating the trap, point the screen up to avoid polyp falling out of trap.
7. Preparation of the retrieved specimen is achieved by removing it from the specimen screen and placing it in the appropriate fixative for histopathology in accordance with institutional guidelines.

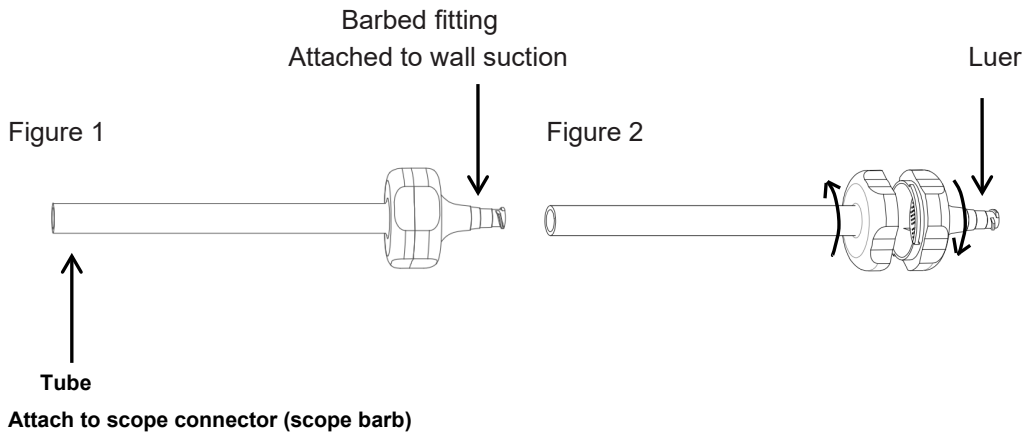

STORAGE

The product should be stored in a cool, dry, clean, well-ventilated, non-corrosive gas environment.

Do not expose the package to organic solvent, ionizing radiation or ultraviolet radiation.

The product shelf life is 3 years.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local and administrative laws and regulations.

Limited Warranty and Disclaimers:

1. Limited Warranty to Buyer. Micro-Tech USA warrants to Buyer that, for the earlier of one (1) year from the date of purchase, or until the product is used by Buyer, the products will be free from defects in materials and workmanship when stored and used in accordance with the instructions for storage and use provided by Micro-Tech USA and in accordance with applicable regulatory requirements. Descriptions or specifications appearing in Micro-Tech USA's literature are meant to generally describe the products and do not constitute any express warranties. In the event that Micro-Tech USA gives technical advice with respect to the product, it is agreed that such advice is given without any liability on Micro-Tech USA's part. Any guarantee of specific properties of or in the products shall only be effective if and to the extent specifically confirmed by Micro-Tech USA in writing. These warranties shall not apply for product failure or deficiency due to improper storage, alteration, or the consequences of uses for which the products were not designed or that adversely affect the products' integrity, reliability, or performance.
2. Disclaimer and Release. **THE WARRANTIES, OBLIGATIONS, AND LIABILITIES OF Micro-Tech USA AS SET FORTH HEREIN ARE EXCLUSIVE. BUYER HEREBY WAIVES ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO THE PRODUCTS AND ANY OTHER GOODS OR SERVICES DELIVERED BY BUYER, INCLUDING, BUT NOT LIMITED TO: (1) ALL OTHER EXPRESS AND IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND (2) ANY IMPLIED WARRANTY ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING, OR USAGE OF TRADE.**
3. Implied Warranties. The purchase of products may be subject to laws in the territories applicable to the sale of the products by Micro-Tech USA to Buyer that impose implied warranties, conditions, or obligations that cannot be excluded, restricted, or modified, or can only be excluded, restricted, or modified to a limited extent. The provisions of Paragraphs 2 and 4 shall apply to the greatest extent allowed by such laws.
4. Limitation of Liability. **EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, Micro-Tech USA'S LIABILITY UNDER THIS WARRANTY IS LIMITED TO: (a) THE REPLACEMENT OF THE PRODUCTS OR THE RE-SUPPLY OF EQUIVALENT PRODUCTS; (b) THE REPAIR OF THE PRODUCTS OR PAYMENT OF THE COST OF REPAIRING THE PRODUCTS; or (c) PAYMENT OF THE COST OF REPLACING THE PRODUCTS OR ACQUIRING EQUIVALENT PRODUCTS. MICRO-TECH USA SHALL HAVE NO OBLIGATION OR LIABILITY, WHETHER ARISING IN CONTRACT (INCLUDING WARRANTY), TORT (INCLUDING ACTIVE, PASSIVE, OR IMPUTED NEGLIGENCE, STRICT LIABILITY, OR PRODUCT LIABILITY) OR OTHERWISE, FOR ANY SPECIAL, CONSEQUENTIAL, PUNITIVE, INCIDENTAL, OR INDIRECT DAMAGES, OR FOR LOSS OF USE, LOSS OF REVENUE, LOSS OF BUSINESS, LOST PROFIT, OR OTHER FINANCIAL LOSS ARISING OUT OF OR IN CONNECTION WITH ANY PRODUCT OR OTHER GOODS OR SERVICES FURNISHED BY MICRO-TECH USA, EVEN IF MICRO-TECH USA WAS AWARE OF THE POSSIBILITY OF SUCH DAMAGES OR LOSS.**



Manufactured for

Micro-Tech USA Inc.

2855 Boardwalk Drive, Ann Arbor, MI 48104 USA

Tel: 734-259-3768

Toll free: 877-552-4027

info@micro-tech-usa.com

Made in P.R. China

Rev: 2
2019.08