



## ESD Specimen Tray Instructions for Use

Consult Instructions for Use

Do Not Reuse

Do Not Use if Package is Damaged

Non-Sterile      Rx only

### **IMPORTANT INFORMATION**

Caution: Federal law restricts this device to sale by or on the order of a physician. Read all instructions carefully before use. They contain essential information on using this device safely and effectively. Keep these instructions in a safe, accessible location, as you may need to refer to them again. If you have any questions or comments about any information in these instructions, please contact Micro-Tech USA.

### **INTENDED USE**

To store and transport specimens.

### **CONTRAINDICATIONS**

- There are no known contraindications.

### **WARNINGS AND PRECAUTIONS**

- The product is intended for single use only! DO NOT re-use, and/or reprocess. Re-use, or reprocessing may compromise the structural integrity of the device and/or lead to failure. Micro-Tech assumes no liability with respect to instruments reused, or reprocessed.
- After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local and administrative laws and regulations.
- Do not use this product for any purpose other than the stated intended use. The product is supplied clean and ready for immediate use.
- Do not use the product if the package is damaged.

【 Product Name 】 ESD Specimen Tray

【 Production Date 】 See packaging

【 Shelf Life 】 1 year

Contents supplied non-sterile.

### **PREPARATION**

1. Inspect the package before use for any damage. Do not use if package is damaged. Open the package carefully after verifying the shelf life.
2. If this product shows any signs of damage, do not use. Do not attempt to repair a nonfunctional or damaged product.

### **INSTRUCTIONS FOR USE**

1. Remove ESD Specimen Tray from packaging
2. Open lid of specimen tray by flipping all four of the tabs up.
3. Remove pad and pins from the specimen tray.
4. Use pins to mount the specimen to the pad.

**Note:** The pad is pre-marked indicating distal and proximal orientation. Ensure the specimen is correctly positioned.

5. Add fixative if appropriate following appropriate precautions.
6. Reapply the tray lid and reseal the specimen tray by pressing all four plastic tabs of the lid down until you hear an audible click. Store the tray in an upright position to minimize the risk of leakage.
7. Follow institution guidelines for specimen handling, labelling and transportation.

### **STORAGE**

The product should be stored in a cool, dry, clean, well-ventilated, non-corrosive gas environment. Do not expose the package to organic solvent, ionizing radiation, or ultraviolet radiation.

### **PRODUCT DISPOSAL**

After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local and administrative laws and regulations.

### **Limited Warranty and Disclaimers**

1. Limited Warranty to Buyer. Micro-Tech USA warrants to Buyer that, for the earlier of one (1) year from the date of purchase, or until the product is used by Buyer. The products will be free from defects in materials and workmanship when stored and used in accordance with the instructions for storage and use provided by Micro-Tech USA and in accordance with applicable regulatory requirements.

Descriptions or specifications appearing in Micro-Tech USA's literature are meant to generally describe the products and do not constitute any express warranties.

In the event that Micro-Tech USA gives technical advice with respect to the product, it is agreed that such advice is given without any liability on Micro-Tech USA's part. Any guarantee of specific properties of or in the products shall only be effective if and to the extent specifically confirmed by Micro-Tech USA in writing. These warranties shall not apply for product failure or deficiency due to improper storage, alteration, or the consequences of uses for which the products were not designed or that adversely affect the products' integrity, reliability, or performance.

2. Disclaimer and Release. THE WARRANTIES, OBLIGATIONS, AND LIABILITIES OF Micro-Tech USA AS SET FORTH HEREIN ARE EXCLUSIVE. BUYER HEREBY WAIVES ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO THE PRODUCTS

AND ANY OTHER GOODS OR SERVICES DELIVERED BY BUYER, INCLUDING, BUT NOT LIMITED TO: (1) ALL OTHER EXPRESS AND IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND (2)

ANY IMPLIED WARRANTY ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING, OR USAGE OF TRADE.

3. Implied Warranties. The purchase of products may be subject to laws in the territories applicable to the sale of the products by Micro-Tech USA to Buyer that impose implied warranties, conditions, or obligations that cannot be excluded, restricted, or modified, or can only be excluded, restricted, or modified to a limited extent. The provisions of Paragraphs 2 and 4 shall apply to the greatest extent allowed by such laws.

4. Limitation of Liability. Except to the extent prohibited by applicable law, Micro-Tech USA's liability under this warranty is limited to: (a) the replacement of the products or the resupply of equivalent products; (b) the repair of the products or payment of the cost of repairing the products; or (c) payment of the cost of replacing the products or acquiring equivalent products. Micro-Tech USA shall have no obligation or liability, whether arising in contract (including warranty), tort (including active, passive, or imputed negligence, strict liability, or product liability) or otherwise, for any special, consequential, punitive, incidental, or indirect damages, or for loss of use, loss of revenue, loss of business, lost profit, or other financial loss arising out of or in connection with any product or other goods or services furnished by Micro-Tech USA, even if Micro-Tech USA was aware of the possibility of such damages or loss.

**Manufactured for:**



**Micro-Tech USA Inc.**  
2855 Boardwalk Drive,  
Ann Arbor, MI 48104 USA  
Tel: 734-259-3768  
Toll free: 877-552-4027  
[info@micro-tech-usa.com](mailto:info@micro-tech-usa.com)  
Made in P.R. China

[SDST3A0010A](#) Rev: 1

2021.01.28