



SURECLIP[®]


PROCEDURAL STEPS FOR HEMOSTATIC CLIP

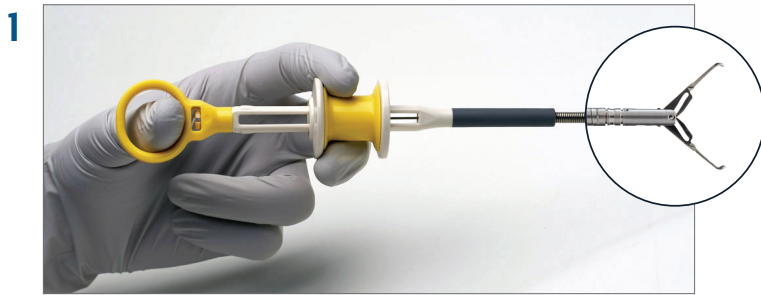


Ensure the clip is closed, carefully insert SureClip into the endoscope through the working channel with a diameter of 2.8mm or larger. When SureClip is visible exiting the distal end of the scope, advance it towards the targeted site. Then open clip by advancing the finger spool forward.

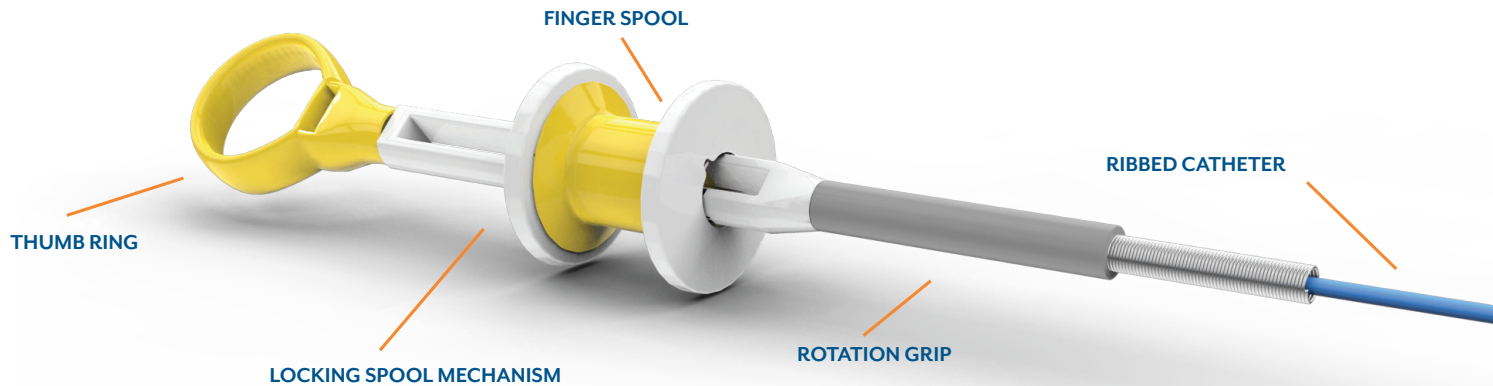

The clip can be rotated clockwise and counterclockwise by slowly turning the grey rotational grip. During rotation, the finger spool must be allowed to rotate freely, remove fingers from the finger spool if necessary.

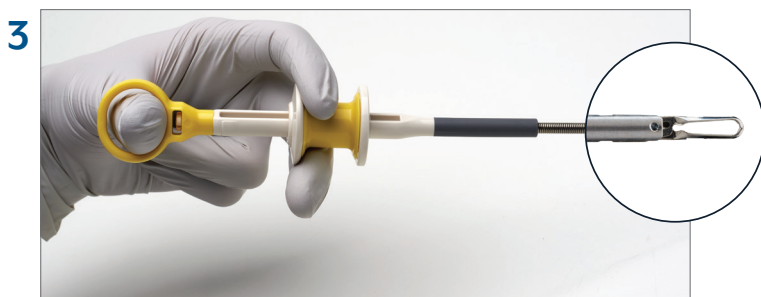

To close the clip, pull the finger spool back until slight resistance is felt in the handle. SureClip can be repositioned by opening and closing the clip as many times as necessary to achieve the ideal placement. Note: Do not forcefully push into the tissue while closing.

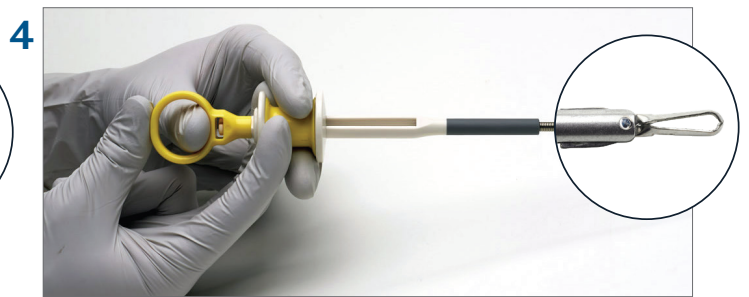

To deploy the clip, continue pulling back with the finger spool towards the thumb ring. Two handed deployment is recommended. You will hear and feel a click when the clip is deployed. The finger spool will lock into place and should not be moved.



This procedure guide is a brief summary and intended for illustrative purposes only. Please refer to the Instructions For Use for complete instructions. SureClip is a trademark of Micro-Tech Endoscopy or its affiliates. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with the product. CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.



EXCEPTIONAL
ENDOSCOPIC
SOLUTIONS.

2855 Boardwalk Drive
Ann Arbor, MI 48104
877.552.4027

MTENDOSCOPY.COM

CustomerCare@MTEndoscopy.com
MPW30000PG Rev.1