



Medical Device Organizer Instructions for Use

 Consult Instructions for Use  Do Not Reuse

 Non-Sterile  **R_x only**

 Do Not Use If Package Is Damaged

IMPORTANT INFORMATION

Caution: Federal law restricts this device to sale by or on the order of a physician. Read all instructions carefully before use. They contain essential information on using this device safely and effectively. Keep these instructions in a safe, accessible location as you may need to refer to them again. If you have any questions or comments about any information in these instructions, please contact Micro-Tech USA.

INTENDED USE

The Micro-Tech Medical Device Organizer is intended to hold various devices used throughout endoscopic procedure(s). It offers five oversized pouches with accompanying hooks to help maintain storage and organization of the various devices used throughout endoscopic procedure(s).

WARNINGS AND PRECAUTIONS

- The product is intended for single use only! DO NOT re-use, and/or reprocess. Re-use, or reprocessing may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Re-use, or reprocessing may also create a risk of contamination of the device and/or cause patient infectious disease(s). Contamination of the device may lead to injury, illness or death of the patient. Micro-Tech assumes no liability with respect to instruments reused, or reprocessed.
- Do not use this device for any purpose other than its intended use.
- The product is only intended for adult populations.
- Devices are not included.
- This product contains five plastic pouches. It should never be used underneath or covering a patient's head as suffocation could result.
- Avoid product contact with direct electrical current (e.g., hot snares).

【 Product Name 】 Medical Device Organizer

【 Production Date 】 See packaging

【 Shelf Life 】 5 years

This device is not made with natural rubber latex.

PREPARATION

1. Inspect the package before use for any damage. Do not use if package is damaged.
2. Verify the expiration date. Do not use if expired.
3. Open the package and remove the organizer.

INSTRUCTIONS FOR USE

1. Open the pouch and unfold the organizer.
2. Peel the adhesive tape tab from the backside of the device and secure the medical device organizer to the desired surface and location (such as countertop, endoscopic cart, beneath the gurney mattress, or over the gurney's side rail).
3. Place endoscopic devices into any or all of the five pouches throughout the procedure.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local and administrative laws and regulations.

STORAGE

The product should be stored in a cool, dry, clean, well-ventilated, non-corrosive gas environment. Do not expose the package to organic solvent, ionizing radiation or ultraviolet radiation.

LIMITED WARRANTY AND DISCLAIMERS

1. Limited Warranty to Buyer. Micro-Tech USA warrants to Buyer that, for the earlier of one (1) year from the date of purchase, or until the product is used by Buyer, the products will be free from defects in materials and workmanship when stored and used in accordance with the instructions for storage and use provided by Micro-Tech USA and in accordance with applicable regulatory requirements. Descriptions or specifications appearing in Micro-Tech USA's literature are meant to generally describe the products and do not constitute any express warranties. In the event that Micro-Tech USA gives technical advice with respect to the product, it is agreed that such advice is given without any liability on Micro-Tech USA's part. Any guarantee of specific properties of or in the products shall only be effective if and to the extent specifically confirmed by Micro-Tech USA in writing. These warranties shall not apply for product failure or deficiency due to improper storage, alteration, or the consequences of uses for which the products were not designed or that adversely affect the products' integrity, reliability, or performance.
2. Disclaimer and Release. The warranties, obligations, and liabilities of Micro-Tech USA as set forth herein are exclusive. Buyer hereby waives all other warranties, express or implied, arising by law or otherwise, with respect to the products and any other goods or services delivered by buyer, including, but not limited to: (1) all other express and implied warranties,

including any implied warranty of merchantability or fitness for a particular purpose, and (2) any implied warranty arising from course of performance, course of dealing, or usage of trade.

3. Implied Warranties. The purchase of products may be subject to laws in the territories applicable to the sale of the products by Micro-Tech USA to Buyer that impose implied warranties, conditions, or obligations that cannot be excluded, restricted, or modified, or can only be excluded, restricted, or modified to a limited extent. The provisions of Paragraphs 2 and 4 shall apply to the greatest extent allowed by such laws.
4. Limitation of Liability. Except to the extent prohibited by applicable law, Micro-Tech USA's liability under this warranty is limited to: (a) the replacement of the products or the re-supply of equivalent products; (b) the repair of the products or payment of the cost of repairing the products; or (c) payment of the cost of replacing the products or acquiring equivalent products. Micro-Tech USA shall have no obligation or liability, whether arising in contract (including warranty), tort (including active, passive, or imputed negligence, strict liability, or product liability) or otherwise, for any special, consequential, punitive, incidental, or indirect damages, or for loss of use, loss of revenue, loss of business, lost profit, or other financial loss arising out of or in connection with any product or other goods or services furnished by Micro-Tech USA, even if Micro-Tech USA was aware of the possibility of such damages or loss.



Manufactured for
Micro-Tech USA Inc.
2855 Boardwalk Drive
Ann Arbor, MI 48104 USA
Tel: 734-259-3768
Toll free: 877-552-4027
info@micro-tech-usa.com
Made in P.R. China