

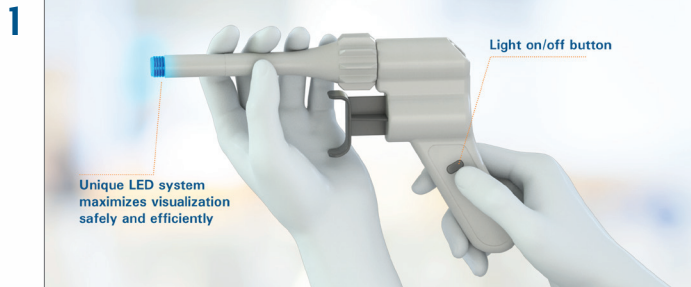


# IN-SIGHT™ BANDER


## PROCEDURAL STEPS FOR IN-SIGHT MULTI-BAND LIGATOR

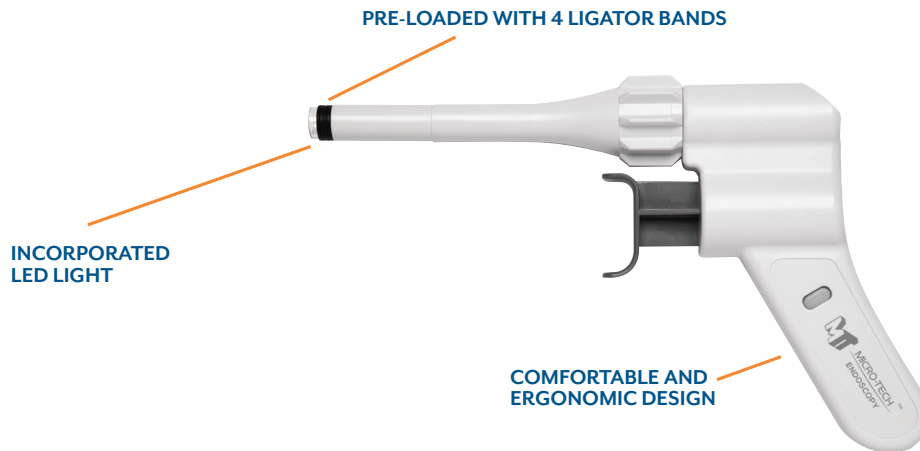


Turn the light on and check the suction by pulling and releasing the trigger. Move the bands to the front of the suction chamber by rotating the actuation knob in a clockwise motion. Direct visualization is possible by looking through the barrel of the device. This is important because ligation bands must be properly placed about 10 millimeters above the dentate line.

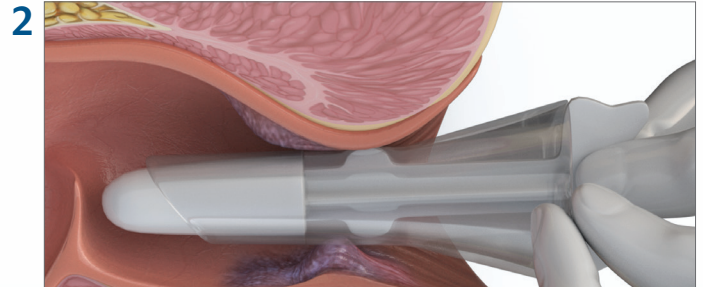

To start, perform a rectal examination and insert the anoscope. Remove the obturator and slightly pull the anoscope back to get the hemorrhoids in view. Reinsert the anoscope at the upper border of the hemorrhoids. Tilt the anoscope towards the target tissue.

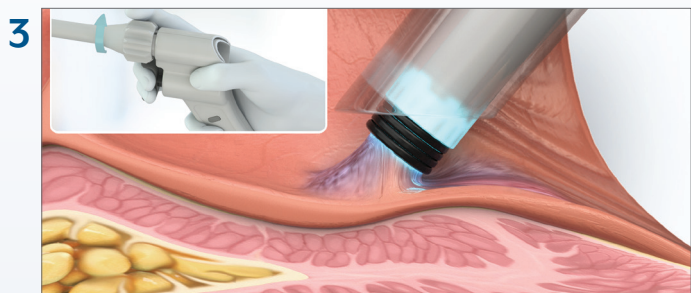

Squeeze the trigger to create suction and approximate the tip of the ligator towards the target area. Release the trigger; this will suction the tissue inside the barrel. Turn the actuation knob clockwise to release the outermost band.

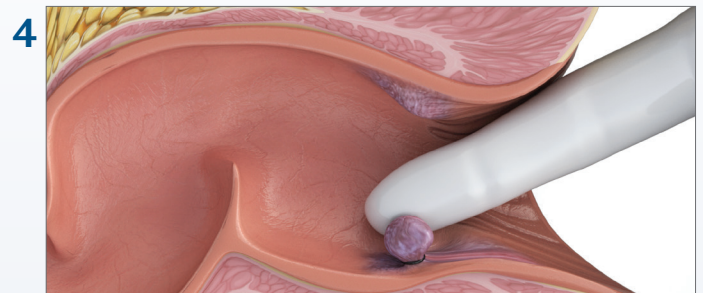

Up to three ligations can be performed under one treatment session without removing the anoscope from the patient. Once you do remove the anoscope, perform rectal examination to confirm proper rubber band placement by palpating the narrow neck of the banded hemorrhoid.

This procedure guide is a brief summary and intended for illustrative purposes only. Please refer to the Instructions For Use for complete instructions. In-Sight is a trademark of Micro-Tech Endoscopy or its affiliates. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with the product.  
CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.