


CLIP WITH CONFIDENCE THE SURECLIP



SURECLIP

Shortest stem, approximately 5 mm,
for use in narrow lumen

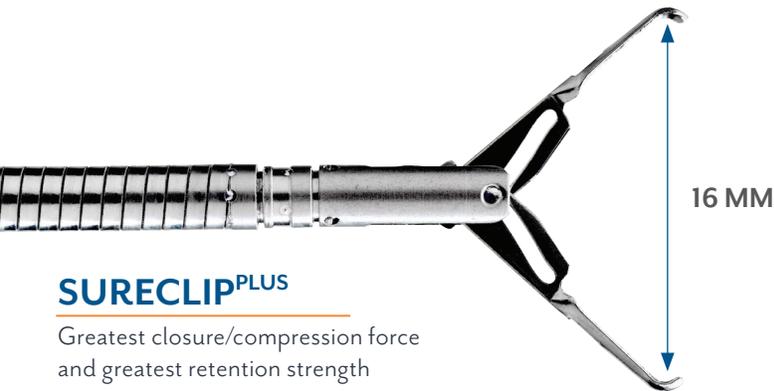


SURECLIP^{PLUS}

Greatest closure/compression force
and greatest retention strength



SURECLIP^{MAX}

Widest jaw opening to accommodate
greater tissue volume

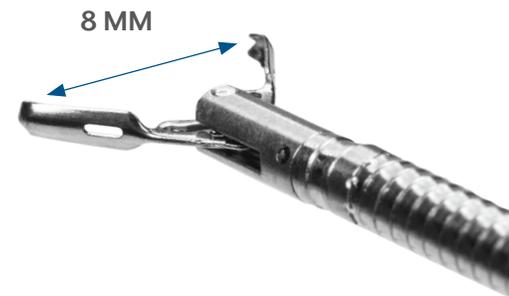

SURECLIP^{MINI}

Lowest jaw profile for unobstructed
view during placement

Clips need to be reliable. They need to be accurate. They need to allow you the flexibility to reposition or rotate as much as is required to deliver better outcomes.

Accurate positioning prior to deployment can reduce both procedure time and the number of clips needed to achieve hemostasis. SureClip[®] achieves this by design delivering outstanding repositionability and reliable rotation prior to deployment, in various scope positions. SureClip's short stem aids placement in narrow lumen and improves visibility.

The proprietary clip design provides reliable deployment, may improve retention, and offers a choice of jaw sizes.

KEY BENEFITS

REPOSITIONING

SureClip's unique design permits opening and closing the jaw prior to deployment. Being able to reposition a clip may help improve placement accuracy. Fenestrations on the SureClip PLUS, SureClip MINI and SureClip MAX accommodate tissue and may enhance retention.

RELIABLE ROTATION

SureClip can be rotated, helping to provide the correct orientation for tissue approximation or defect closure. The rotation handle on the SureClip improves performance and enhances the user experience.

SHORT STEM

A shorter stem makes the clip less obtrusive, improving visualization of the target area, particularly when multiple clips are placed in close proximity.

SURECLIP^{MINI} LOWEST PROFILE STERILE



Order Number	Opening Width (mm)	Sheath Diameter (mm)	Working Length (cm)	Minimum Channel Size (mm)	Package Units
RC30415	8	Max 2.6	235	2.8	2/Box
RC30411	8	Max 2.6	235	2.8	10/Box

SURECLIP SHORTEST STEM STERILE



Order Number	Opening Width (mm)	Sheath Diameter (mm)	Working Length (cm)	Minimum Channel Size (mm)	Package Units
RC30445	11	Max 2.6	235	2.8	2/Box
RC30441	11	Max 2.6	235	2.8	10/Box

SURECLIP^{PLUS} GREATEST CLOSURE/COMPRESSION FORCE STERILE



Order Number	Opening Width (mm)	Sheath Diameter (mm)	Working Length (cm)	Minimum Channel Size (mm)	Package Units
RC30385	16	Max 2.6	235	2.8	2/Box
RC30381	16	Max 2.6	235	2.8	10/Box

SURECLIP^{MAX} WIDEST JAW OPENING* STERILE



Order Number	Opening Width (mm)	Sheath Diameter (mm)	Working Length (cm)	Minimum Channel Size (mm)	Package Units
RC30545	17	Max 2.6	235	2.8	2/Box
RC30541	17	Max 2.6	235	2.8	10/Box

*Jaws may not open to maximum reach until placed in contact with tissue



2855 Boardwalk Drive
Ann Arbor, MI 48104
877.552.4027

MTEndoscopy.com
customer@mtendoscopy.com

MPW30000 Rev.10