


LOCK ONTO TISSUE

LOCKADO HEMOSTASIS CLIP

Hemostasis clips need to provide accuracy, reliability and precise functionality. Lockado[®] allows unrestricted opening and closing so it can be repositioned until the desired position is achieved.

With two sets of opposing atraumatic teeth in the arms of the clip, Lockado delivers a reliable and secure grip for consistent performance. Driven by Micro-Tech Endoscopy's proven clip delivery system, the Lockado clip provides exceptional reliability and tissue management.

- Jaw style works for multiple clinical applications
- Can be used with side-viewing endoscope
- Shorter stem makes the clip less obtrusive
- Variety of sizes: 11 mm, 16 mm and MAX 22 mm arm opening widths
- Indicated for anchoring jejunal feeding tubes

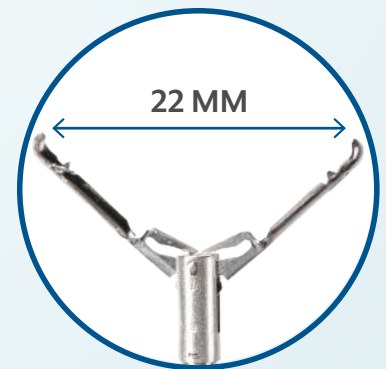

KEY FEATURES

ATRAUMATIC TEETH

6.3 MM / 6.3 MM / 5.3 MM

14.9 MM / 12.9 MM / 10.6 MM

SPOOL

Familiar design for precise actuation of jaws and clip deployment.

THUMB RING

Ergonomic thumb ring design allows for one or two-handed clip deployment.

LOCKING SPOOL MECHANISM

Mechanism provides tactile and audible feedback of clip deployment. Spool is locked in place and clip is released automatically.

RIBBED CATHETER

The lubricious ribbed catheter surrounds a torque wire that provides 1:1 response and 360° rotation without inadvertent clip deployment or helicoptering.

ROTATION GRIP

Secure grip allows for 360° bidirectional controlled rotation and placement of clip on targeted tissue.

SPECIFICATIONS

LOCKADO HEMOSTASIS CLIP STERILE

Order Number	Opening Width (mm)	Sheath Diameter (mm)	Working Length (cm)	Minimum Channel Size (mm)	Packing Units
LO30861	11	2.6	235	2.8	10/Box
LO30865	11	2.6	235	2.8	2/Box

LOCKADO PLUS HEMOSTASIS CLIP STERILE

Order Number	Opening Width (mm)	Sheath Diameter (mm)	Working Length (cm)	Minimum Channel Size (mm)	Packing Units
LO30891	16	2.6	235	2.8	10/Box
LO30895	16	2.6	235	2.8	2/Box

LOCKADO MAX HEMOSTASIS CLIP* STERILE

Order Number	Opening Width (mm)	Sheath Diameter (mm)	Working Length (cm)	Minimum Channel Size (mm)	Packing Units
LO35041	22	2.6	235	2.8	10/Box
LO35045	22	2.6	235	2.8	2/Box

*Jaws may not open to maximum reach until placed in contact with tissue.

CAN'T FIND IT?

Additional items may be available. Contact us if you can't find what you need.