## **Single-use Biopsy Forceps**

Instructions for Use

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Mode d'emploi

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## IMPORTANT INFORMATION

Caution: Federal law restricts this device to sale by or on the order of a physician. Read all instructions carefully before use. They contain essential information on using this device safely and effectively. Keep these instructions in a safe, accessible location, as you may need to refer to them again. If you have any questions or comments about any information in these instructions, please contact Micro-Tech.

EN

## INTENDED USE

The Single-use Biopsy Forceps are used for endoscopic histological sampling of various tissue sites within the gastrointestinal tracts via the operating channel of endoscopic instruments. A thorough understanding of the technical principles, clinical applications, and risks associated with gastrointestinal endoscopy, mucosal biopsy and foreign body retrieval is necessary before using this device. Biopsy forceps should only be used by, or under the supervision of, physicians thoroughly trained in therapeutic endoscopy, mucosal biopsy and foreign body retrieval.

## PATIENT TARGET GROUP

The patient population of this device is shown in the following table.

REF	Patient Target Group
NBF03-11018160	Adult, Adolescent and Child over 2 years old and weighing no less than 3.5Kg.
NBF03-11118160	
NBF13-11018160	
NBF13-11118160	
Others	Adult

## CONTRAINDICATIONS

The contraindications of the primary endoscopic procedure are included. Usually listed in this way:

1. Coagulopathy
2. Tissue samples from vessel injury in ulceration
3. Insufficient prepped bowel
4. Those specific to the primary endoscopic procedure.
5. Use in children under 2 years of age or weighing less than 3.5Kg.(This rule applies only to pediatrics)

## POTENTIAL COMPLICATIONS

---

1. Perforation
2. Hemorrhage
3. Infection
4. Sepsis
5. Allergic reaction to contrast or medication
6. Hypotension
7. Respiratory depression or arrest
8. Cardiac arrhythmia or arrest
9. Affect the mental health of children  
(This rule applies only to pediatrics)

## WARNINGS

---

1. **The product is intended for single use only! DO NOT re-use, re-sterilize, and/or reprocess.** Re-use, re-sterilization or reprocessing may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Re-use, re-sterilization or reprocessing may also create a risk of contamination of the device and/or cause patient infectious disease(s). Contamination of the device may lead to injury, illness or death of the patient. Micro-Tech assumes no liability with respect to instruments reused, re-sterilized or reprocessed .
2. **Do not use this instrument for any purpose other than its intended use.**
3. This device is not made with natural rubber latex.
4. Patients should be informed of the potential risks and complications, which may lead to injury, illness or death of the patient.
5. **The instrument is intended for use under the direct supervision of a suitably trained physician only.** A thorough understanding of the technical principles, clinical applications, and associated risks is expected before usage.
6. Confirm that the endoscopy view is clear before use. Do not insert the instrument into the endoscope unless you have a clear endoscopic field of view. Insertion without clear endoscopic field of view could cause patient injury, such as perforation, hemorrhage or mucous membrane damage. Damage to the endoscope and/or the instrument may also occur.
7. If the sample is too large to close the forceps cups sufficiently, withdraw the forceps to the tip of the endoscope and then withdraw the endoscope and biopsy forceps simultaneously.
8. Should there occur any irregularities, dispose the instrument and replace it with a new one.

- [ Product Name ] Single-use Biopsy Forceps
- [ Packaging ] Packed in Pouch
- [ Production Date ] See packaging
- [ Sterilization ] Sterilized by EO (ethylene oxide) gas
- [ Shelf Life ] See packaging
- [ Compatible Working Channel ]

External diameter of Jaws $\varnothing$ (mm)	working channel $\varnothing$ (mm)
1.8	$\geq 2.0$
2.2	$\geq 2.8$
2.3	
2.4	
2.8	$\geq 3.2$
3.0	$\geq 3.7$
3.3	

## PREPARATION

1. Reference the product label and choose the appropriate device.
2. Contents supplied STERILE.
3. Inspect the package before use for any damage. Do not use if package is damaged. Open the package carefully after verifying the shelf life.
4. Carefully remove the device from its packaging and uncoil it. Do NOT use excessive force as this may damage the device and affect performance.
5. Before use, check the clip and the spring tube to ensure that there are no sharp edges.
6. If this device shows any signs of damage, do not use. Do not attempt to repair a nonfunctional or damaged device.

## INSTRUCTIONS FOR USE

1. Visually inspect the biopsy forceps with particular attention to bends or breaks. If it appears damaged, do not use the device.
2. Perform the diagnostic endoscopy and visualize the area to be biopsied Insert the biopsy forceps with the cups closed into the accessory channel in small increments.
3. Open the cups and advance the device into the tissue to be biopsied.
4. Close the biopsy forceps cups using slight pressure on the handle. Maintain gentle handle pressure to keep the cups closed while gently withdrawing the forceps from the tissue.
5. Withdraw the biopsy forceps from the channel. Continue to apply slight pressure on the handle to hold the cups closed.
6. Prepare the specimen for examination per institutional guidelines.
7. Dispose of for single use only products according to the guidelines for biohazardous medical waste.

## STORAGE

---

The product should be stored in a cool, dry, clean, well-ventilated, non-corrosive gas environment.

Do not expose the package to organic solvent, ionizing radiation or ultraviolet radiation.

## PRODUCT DISPOSAL

---

After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local and administrative laws and regulations.

## Limited Warranty and Disclaimers

---

1. Limited Warranty to Buyer. Micro-tech USA warrants to Buyer that, for the earlier of one (1) year from the date of purchase, or until the product is used by Buyer, the products will be free from defects in materials and workmanship when stored and used in accordance with the instructions for storage and use provided by Micro-tech USA and in accordance with applicable regulatory requirements. Descriptions or specifications appearing in Micro-tech USA's literature are meant to generally describe the products and do not constitute any express warranties. In the event that Micro-tech USA gives technical advice with respect to the product, it is agreed that such advice is given without any liability on Micro-tech USA's part. Any guarantee of specific properties of or in the products shall only be effective if and to the extent specifically confirmed by Micro-tech USA in writing. These warranties shall not apply for product failure or deficiency due to improper storage, alteration, or the consequences of uses for which the products were not designed or that adversely affect the products' integrity, reliability, or performance.
2. Disclaimer and Release. **THE WARRANTIES, OBLIGATIONS, AND LIABILITIES OF Micro-tech USA AS SET FORTH HEREIN ARE EXCLUSIVE. BUYER HEREBY WAIVES ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO THE PRODUCTS AND ANY OTHER GOODS OR SERVICES DELIVERED BY BUYER, INCLUDING, BUT NOT LIMITED TO: (1) ALL OTHER EXPRESS AND IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND (2) ANY IMPLIED WARRANTY ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING, OR USAGE OF TRADE.**
3. Implied Warranties. The purchase of products may be subject to laws in the territories applicable to the sale of the products by Micro-tech USA to Buyer that

impose implied warranties, conditions, or obligations that cannot be excluded, restricted, or modified, or can only be excluded, restricted, or modified to a limited extent. The provisions of Paragraphs 2 and 4 shall apply to the greatest extent allowed by such laws.

4. **Limitation of Liability. EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, MICRO-TECH USA'S LIABILITY UNDER THIS WARRANTY IS LIMITED TO: (a) THE REPLACEMENT OF THE PRODUCTS OR THE RE-SUPPLY OF EQUIVALENT PRODUCTS; (b) THE REPAIR OF THE PRODUCTS OR PAYMENT OF THE COST OF REPAIRING THE PRODUCTS; or (c) PAYMENT OF THE COST OF REPLACING THE PRODUCTS OR ACQUIRING EQUIVALENT PRODUCTS. MICRO-TECH USA SHALL HAVE NO OBLIGATION OR LIABILITY, WHETHER ARISING IN CONTRACT (INCLUDING WARRANTY), TORT (INCLUDING ACTIVE, PASSIVE, OR IMPUTED NEGLIGENCE, STRICT LIABILITY, OR PRODUCT LIABILITY) OR OTHERWISE, FOR ANY SPECIAL, CONSEQUENTIAL, PUNITIVE, INCIDENTAL, OR INDIRECT DAMAGES, OR FOR LOSS OF USE, LOSS OF REVENUE, LOSS OF BUSINESS, LOST PROFIT, OR OTHER FINANCIAL LOSS ARISING OUT OF OR IN CONNECTION WITH ANY PRODUCT OR OTHER GOODS OR SERVICES FURNISHED BY MICRO-TECH USA, EVEN IF MICRO-TECH USA WAS AWARE OF THE POSSIBILITY OF SUCH DAMAGES OR LOSS.**

## SYMBOL INDICATIONS

	Do not re-use		Do not re-sterilize
	Date of Manufacture		Manufacturer
	Use-by date		Batch code
	Catalogue Number		Universal Product Number
	Sterilization using ethylene oxide		Keep away from sunlight
	Keep dry		Do not use if package is damaged
	Consult instructions for use or consult electronic instructions for use		Compatible working channel
	Diameter		Working length
	This device does not made with natural rubber latex.		Sterile barrier system/sterile packaging
	Contents		Open here
<b>R<sub>x</sub> only</b>	Federal law prohibits dispensing without prescription		

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**Micro-Tech (Nanjing) Co., Ltd.**
**Micro-Tech USA Inc.**