## **EndoINK™ Endoscopic Marker**

Instructions for Use

## **Marcador endoscópico EndoINK™**

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Prescription only

Consult instructions for use

Sterilized by steam  
Sterile fluid path

Do not resterilize

Do not use if  
package is damaged

Non-pyrogenic

Do not re-use

Use by date

Not made with  
natural rubber latex

Keep away  
from sunlight

Lower limit of  
temperature

Non-sterile protective  
packaging with  
sterile barrier system inside

## IMPORTANT INFORMATION

**Caution:** Federal law restricts this device to sale by or on the order of a physician. Read all instructions carefully before use. They contain essential information on using this device safely and effectively. Keep these instructions in a safe, accessible location, as you may need to refer to them again. If you have any questions or comments about any information in these instructions, please contact Micro-Tech.

## DESCRIPTION

EndoINK™ Endoscopic Marker is designed for endoscopically

marking or “tattooing” lesions in the gastrointestinal tract. EndoINK is a suspension fluid containing high purity carbon black particles. The particles remain in the submucosal injection site creating a permanent mark.

## MAIN COMPONENTS

---

Water for Injection, High Purity Carbon Black, Glycerin, Polysorbate 80, Simethicone, Benzyl Alcohol, Sodium dihydrogen phosphate, Disodium hydrogen phosphate

## INTENDED USE

---

EndoINK™ Endoscopic Marker is indicated for endoscopically marking lesions in the gastrointestinal tract when the endoscopist anticipates the lesion will require surgical removal within 30 days.

## CONTRAINDICATIONS

---

Allergy to device or any of the ingredients in the product. **Not for pediatric use.**

## POTENTIAL COMPLICATIONS

---

Potential complications that are not known or observed.

## ADVERSE REACTIONS

---

Any inadvertent injection of EndoINK™ Endoscopic Marker into the intraperitoneal space should be monitored closely for adverse reactions.

## WARNINGS

---

- For use in the GI tract only – NOT FOR INTRAVACULAR OR INTRAMUSCULAR INJECTION.
- Only the fluid is sterile. Do not use in a sterile or aseptic area without proper precautions.
- **The product is intended for single use only! DO NOT re-use, re-sterilize, and/or reprocess.** Re-use, re-sterilization or reprocessing may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Re-use, re-sterilization or reprocessing may also create a risk of contamination of the device and/ or cause patient infectious disease(s). Contamination of the device may lead to injury, illness or death of the patient. Micro-Tech assumes no liability with respect to instruments reused, re-sterilized or reprocessed.
- Do not use this product for any purpose other than its intended use.
- The product is only intended for adult populations.
- **The product is intended for use under the direct supervision of a trained physician only.** A thorough understanding of the

technical principles, clinical applications, and associated risks is expected before usage.

- Do not use if packaging is damaged.
- The product is to be used in conjunction with endoscopic injection needles, 22-25G are recommended.
- The products compatibility with other substances has not been tested.

**【Product Name】** EndoINK™ Endoscopic Marker

**【Packaging】** Packaged in syringe

**【Production Date】** See packaging

**【Sterilization】** Fluid sterilized by steam

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## PREPARATION

---

1. Reference the product label and choose the appropriate device.
2. Fluid is supplied STERILE.
3. Open the package carefully after verifying the expiration date. CAUTION: For use in the GI tract only – NOT FOR INTRAVACULAR OR INTRAMUSCULAR INJECTION
4. Sterile unless damaged. Leakage of product maybe evidence of damage. Do not use if damaged.

## INSTRUCTIONS FOR USE

---

1. The EndoINK™ Endoscopic Marker syringe should be shaken vigorously until the pigment is fully suspended.
2. Attach the syringe to the Luer fitting on the endoscopic injection needle (not provided with the device). Prime the endoscopic injection needle with EndoINK to expel any air.
3. With the needle retracted, carefully insert the catheter through the biopsy channel of the endoscope under direct visualization until the needle emerges into the endoscopic view.
4. When the needle is properly positioned, insert the tip 30°-45° tangentially into the submucosal space. **CAUTION: Do not insert the needle perpendicular to the colon, as this may lead to perforation of the serosa and injection of the product directly into the peritoneal cavity.**
5. A minimum of three to four injections are recommended to create a circumferential pattern around the lesion. Use 0.5-0.75mL of EndoINK per injection site. The maximum recommended dose is 8mL per patient.

## STORAGE

---

Store the product at room temperature.

## PRODUCT DISPOSAL

---

1. Discard any unused product after the syringe/vial has been opened.
2. After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local, and administrative laws and regulations.

## Limited Warranty and Disclaimers:

---

1. Limited Warranty to Buyer. Micro-Tech USA warrants to Buyer that, for the earlier of one (1) year from the date of purchase, or until the product is used by Buyer, the products will be free from defects in materials and workmanship when stored and used in accordance with the instructions for storage and use provided by Micro-Tech USA and in accordance with applicable regulatory requirements. Descriptions or specifications appearing in Micro-Tech USA's literature are meant to generally describe the products and do not constitute any express warranties. In the event that Micro-Tech USA gives technical advice with respect to the product, it is agreed that such advice is given without any liability on Micro-Tech USA's part. Any guarantee of specific properties of or in the products shall only be effective if and to the extent specifically confirmed by Micro-Tech USA in writing. These warranties shall not apply for product failure or deficiency due to improper storage, alteration, or the consequences of uses

- for which the products were not designed or that adversely affect the products' integrity, reliability, or performance.
2. **Disclaimer and Release. THE WARRANTIES, OBLIGATIONS, AND LIABILITIES OF Micro-Tech USA AS SET FORTH HEREIN ARE EXCLUSIVE. BUYER HEREBY WAIVES ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO THE PRODUCTS AND ANY OTHER GOODS OR SERVICES DELIVERED BY BUYER, INCLUDING, BUT NOT LIMITED TO: (1) ALL OTHER EXPRESS AND IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND (2) ANY IMPLIED WARRANTY ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING, OR USAGE OF TRADE.**
  3. **Implied Warranties.** The purchase of products may be subject to laws in the territories applicable to the sale of the products by Micro-Tech USA to Buyer that impose implied warranties, conditions, or obligations that cannot be excluded, restricted, or modified, or can only be excluded, restricted, or modified to a limited extent. The provisions of Paragraphs 2 and 4 shall apply to the greatest extent allowed by such laws.
  4. **Limitation of Liability. EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, Micro-Tech USA' S LIABILITY UNDER THIS WARRANTY IS LIMITED TO: (a) THE REPLACEMENT OF THE PRODUCTS OR THE RE-SUPPLY OF EQUIVALENT PRODUCTS; (b) THE REPAIR OF THE PRODUCTS OR PAYMENT OF THE COST OF REPAIRING THE PRODUCTS; or (c) PAYMENT OF THE COST OF REPLACING THE PRODUCTS OR ACQUIRING EQUIVALENT PRODUCTS. MICRO-TECH USA SHALL HAVE NO OBLIGATION OR LIABILITY, WHETHER**

ARISING IN CONTRACT (INCLUDING WARRANTY), TORT (INCLUDING ACTIVE, PASSIVE, OR IMPUTED NEGLIGENCE, STRICT LIABILITY, OR PRODUCT LIABILITY) OR OTHERWISE, FOR ANY SPECIAL, CONSEQUENTIAL, PUNITIVE, INCIDENTAL, OR INDIRECT DAMAGES, OR FOR LOSS OF USE, LOSS OF REVENUE, LOSS OF BUSINESS, LOST PROFIT, OR OTHER FINANCIAL LOSS ARISING OUT OF OR IN CONNECTION WITH ANY PRODUCT OR OTHER GOODS OR SERVICES FURNISHED BY MICRO-TECH USA, EVEN IF MICRO-TECH USA WAS AWARE OF THE POSSIBILITY OF SUCH DAMAGES OR LOSS.

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**Micro-Tech (Nanjing) Co., Ltd.**
**Micro-Tech USA Inc.**